

CYBERTECH PROTÉGÉ LSO

(FRENCH: TO PROTECT & SUPPORT)

Code Approved L0637

The **ALL NEW Cybertech Protégé LSO** is a multifaceted modular spinal support that takes you from pre-op, initial post-op to a rehab progression stepdown system.

It allows for progressive rehab adaptability with removable/replaceable lateral, posterior and anterior panels for a complete total progressive rehabilitation support system.

The modularity of the system provides 4 levels of support.

Indications:

- Post-operative stabilization protocol following spinal fusion, laminectomy/laminotomy, foraminotomy, laproscopic disk replacement, IDET procedures, multi-level decompression
- Burst Fracture
- Facet syndrome
- Chronic & Mechanical low back pain

Patent Pending

*Removable/replaceable
lateral panels*



Clinical Guidelines:

"CYBERTECH Medical braces are medical products registered with the FDA as a "medical device". CYBERTECH Medical does not prescribe braces or determine the appropriateness of any particular bracing methodology for the individual. Medical advice should be sought on the appropriateness and the condition under which a given brace should be worn."

A general guide to clinical indications for each CYBERTECH Medical spinal orthosis include, but are not limited to, the following:

Indications:

- Post-operative stabilization protocol following spinal fusion, laminectomy/laminotomy, foraminotomy, laproscopic disk replacement, IDET procedures, multi-level decompression
- Burst Fracture
- Facet syndrome
- Chronic & Mechanical low back pain

Fitting Specifications:

Size Circ.	Posterior Panel Ht.	Anterior Panel Ht.	Lateral Panel	Item#:
Small 25"-30"	14"	7 1/4"	Standard	PLSO-NB-1
Medium 30"-35"	14"	7 1/4"	Standard	PLSO-NB-2
Large 35"-40"	15 1/2"	7 1/4"	Large	PLSO-NB-3
X-Large 40"-45"	15 1/2"	7 1/4"	Large	PLSO-NB-4
XX-Large 45"-50"	15 1/2"	9 1/4"	Large	PLSO-NB-5
XXX-Large 50"-55"	15 1/2"	9 1/4"	Large	PLSO-NB-6



Fitting Instructions:

- Fasten the handle near the pulley unit. The handle is always on the left.
- Stand holding the soft side panel apart, centering the back panel on the spine.
- Wrap the left side to the front positioning the front panel..
- Wrap the right side on top of the left panel to close and attach in front. Adjust the front or back panels as needed.
- Pull the handle out and away from the power unit to desired compression and support. Place the handle on the fabric fastener band securely.

Tips:


- Be sure to start the handle near the pulley unit.
- Extend or pull out the brace before application.
- Adjust tension for sitting or standing, experiment to meet your needs.
- Hand wash in warm water, rinse well, towel and air dry.

Posterior and Anterior Panels:

- Dynamic Posterior/Lateral panels conform to each patient's individual body structure for improved comfort and support.
- Unique removable lateral panels.
- Anterior panels are easily attached with integrated pocket w/ Velcro closure allowing additional circumference expansion.

It allows for progressive rehab adaptability with removable lateral panels, posterior and anterior panels for a complete total progressive reduction in rehabilitation protocol. The modularity of the system provides the patient with 4 levels of support.

Patent Pending Brace Design

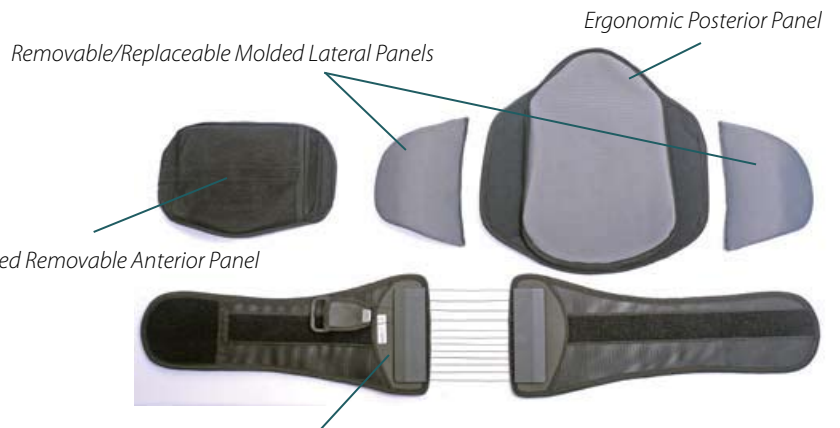

Patented 6 to 1 Mechanical Advantage Pulley System & Belt



1815 Wright Ave., La Verne, CA 91750
Ph: 1.800.220.4224 Fax: 1.909.447.7058
www.cybertechmedical.com

A Bio Cybernetics International Company

CE
Authorized Rep.
RSQA / RSQR Ltd.
www.rsqa.co.uk
info@rsqa.co.uk

Rev. 12/11